# MEMOREG® 2 / MEMOSIL® 2

|    |                            |    |
|----|----------------------------|----|
| DE | Gebrauchsanweisung .....   | 3  |
| GB | Instructions for use ..... | 6  |
| FR | Mode d'emploi .....        | 9  |
| ES | Instrucciones de uso.....  | 12 |
| IT | Istruzioni per l'uso ..... | 15 |
| PT | Instruções de uso .....    | 18 |
| NL | Gebruiksaanwijzing .....   | 21 |
| SE | Bruksanvisning .....       | 24 |
| DK | Brugervejledning .....     | 27 |
| NO | Bruksanvisning .....       | 30 |
| FI | Käyttöohjeet.....          | 33 |
| GR | Οδηγίες χρήσης .....       | 36 |

Heraeus

Heraeus

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# MEMOREG® 2 / MEMOSIL® 2 Instructions for use GB

## MEMOREG® 2

Polyvinyl siloxane for direct application  
very high hardness

### Area of application:

- Standard interocclusal records
- location material for intra-oral pin tracing template

## MEMOSIL® 2

Polyvinyl siloxane for direct application  
transparent

### Area of application:

- Front, occlusal record
- Transfer matrix for brackets (indirect bonding)
- Fixing of x-ray balls when planning the implants

|                         | MEMOREG® 2 | MEMOSIL® 2  |
|-------------------------|------------|-------------|
| Working time*1          | 30 seconds | 30 seconds  |
| Minimum time in mouth*2 | 60 seconds | 3 minutes   |
| Shore A hardness        | 90         | 72          |
| Product colours         | pink       | transparent |

\*1 Total working time (including mixing) at 23 °C/73 °F; shortened to 20s at oral temperature (35 °C/95 °F)

\*2 Minimum time that the bite registration material must be in a closed mouth.

The data given are furnished to the best of our knowledge and are based on internal test results.

### **Instructions for use**

Visually inspect for damage before first use. Damaged products must not be used. Mixing tips and bite registration tips are for single use only. Dispense a small amount of material from a 50 ml cartridge before attaching a mixing tip. Attach mixing tip and fix by rotating clockwise. Dosing and mixing is done automatically on application.

When bite registration materials based on polyvinyl siloxane are touched with latex gloves, retraction solutions or freshly inserted composite fillings, it can change the curing behaviour.

Memoreg 2/Memosil 2 should be used at room temperature (23 °C/73 °F). Lower temperatures, for example when the materials are stored in a refrigerator can extend the working time, and higher temperatures can shorten it.

Hardened polyvinyl siloxanes are chemically resistant. Clothes can be permanently stained if the contact material is not completely cured.

Memoreg 2/Memosil 2 are applied with the green mixing tip (1:1). It can also be combined with the Bite Registration Tip.

### **Disinfection**

Memoreg 2/Memosil 2 can be treated with an aqueous sodium hypochlorite solution (5.25% for up to 10 minutes\*). The treatment does not affect the surface or dimensions.

When using other disinfectant solutions suitable for silicones, follow the disinfection and processing instructions of the manufacturer.

*\*(DGZMK 3/93 V1.0)*

**Storage conditions**

Do not store above 25°C/77°F!

Do not use the material after the expiration date.

Please quote batch number and expiry date in all correspondence about the product.

Expiration date  and batch number ; See note on packaging.

**Warning and safety instructions**

If the material contacts your eyes, rinse with copious amounts of water for several minutes. If symptoms persist, consult an eye specialist.

Do not inhale, swallow or ingest. If health problems arise after swallowing impression material, seek medical attention immediately.

Intestinal blockage may arise in rare cases. Immediately consult a doctor if the product is inhaled.

Do not leave any residual material in the sulcus or oral cavity.

Memoreg 2 should not be used for conventional impressions since its high final hardness makes it very difficult to remove from the mouth. Undesired tooth extractions may result. Hypersensitive persons may experience an allergic reaction to the product.

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Heraeus Kulzer GmbH
Heraeus Kulzer, LLC