1 -STONE III

- EN** Synthetic Dental Hard Stone Type III
LT Sintetinis, kietas odontologinis gipsas, Tipas III
DA Syntetisk Dental Hård Sten Type III
NO Type III syntetisk tannhard stein
SV Syntetisk dental hård sten typ III

ISO 6873, Type 3 / ISO 6873, Tipas 3/ ISO 6873, Type 3/ ISO 6873, Type 3/ ISO 6873, Typ 3

INSTRUCTION FOR USE EN

PHYSICAL DATA

Mixing ratio	powder 100g / distilled water 30ml
Setting time	app. 15 min.
Setting expansion after 2h	max. 0.2 %
Compressive strength after 1h	min. 20 MPa

INSTRUCTIONS FOR USE

For mixing use a metal or plastic spatula and clean mixing bowl. Strew the powder into the water within 10 seconds and allow the mix to soak for 20 seconds. Mixing time by hands is 60 seconds. Mechanical mixing time is 30 seconds. The initial consistency becomes more fluid during mixing and reaches optimum fluidity, when pouring the moulds on the vibrator. Pouring time is app. 6 minutes. Do not vibrate after this time so as not to disturb the crystallization.

STORAGE

Store well closed in cool and dry place. Seal well between usages. Keep at approx. 23°C for several hours before use. Gypsum products are subject to deterioration when exposed to the atmosphere, particularly if the humidity is high.

MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

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REF IS3TPY	5000g yellow
REF IS3CPY	25000g yellow
REF IS3TPG	5000g green
REF IS3CPG	25000g green
REF IS3TPB	5000g blue
REF IS3CPB	25000g blue

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