Directions For Use

Identium®

Vinylsiloxanether® Impression Material ISO 4823

Made in Germany
37402/1010

Inhalt/Content/Contenu/Contenuto/Contenido/Conteúdo/содержимое/Svenska

Deutsch	3	Español	51
Produktbeschreibung	4	Descripción del producto	52
Produktüberblick	7	Descripción general de productos	55
Technische Daten	8	Datos técnicos	56
Legende	14	Leyenda	62
English	15	Português	63
Product Description	16	Descrição do produto	64
Product Overview	19	Vista geral dos produtos	67
Technical Data	20	Dados técnicos	68
Legend	26	Legenda	74
Français	27	Русско	75
Description du produit	28	Описание продукта	76
Aperçu du produit	31	Обзор материалов	79
Caractéristiques techniques	32	Технические характеристики	80
Légende	38	Подписи к рисункам	85
Italiano	39	Svenska	87
Descrizione del prodotto	40	Produktbeskrivning	88
Panoramica dei prodotti	43	Produktinformation	91
Specifiche tecniche	44	Teknisko data	92
Legenda	50	Teckenförklaring	98

- Identium[®]** Heavy
- Identium[®]** Heavy fast
- Identium[®]** Medium
- Identium[®]** Medium fast
- Identium[®]** Medium soft
- Identium[®]** Medium soft fast
- Identium[®]** Light
- Identium[®]** Light fast

Hersteller:

Kettenbach GmbH & Co. KG

Im Heerfeld 7

35713 Eschenburg, Germany

www.kettenbach.com

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Kettenbach LP

7777 Center Avenue, Suite 280

Huntington Beach, CA 92647, USA

Vinylsiloxanether[®] Impression Material ISO 4823

Made in Germany

37402/1010

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Vinylsiloxanether[®] Impression Material ISO 4823

Made in Germany
37402/1010

For professional use only. Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a dentist.

Vinylsiloxanether® Impression Material ISO 4823

Product Description

Identium® products are addition-curing, elastomeric impression materials with parts of vinylpolysiloxane and polyether (Vinylsiloxanether®). They convince with excellent flow and hydrophilic properties, high tear strength, dimensional accuracy, and resistance to permanent deformation.

The Identium® product range consists of three different viscosities (heavy-bodied, medium-bodied, light-bodied), available in two delivery systems: standard 1:1 (50 ml automix cartridges) and 5:1 (362 ml foil bags) for use in most automatic dispensing and mixing systems. The Identium® products are all available in a regular and fast-set version.

Indications/Techniques

Identium® Heavy/Identium® Heavy fast is to be used as a heavy-bodied material in one-step technique (double mix) for:

- Impressions for crowns/bridges, inlays/onlays and veneer preparations

- Functional impressions
- Impressions for full or partial dentures
- Implant impressions

Identium® Medium/Identium® Medium fast is to be used as a medium-bodied tray or syringeable impression material in one-step technique (monophase or double mix) for:

- Impressions for crowns/bridges, inlays/onlays and veneer preparations
- Implant impressions
- Fixation impressions
- Functional impressions

Identium® Medium soft/Identium® Medium soft fast is to be used as a medium-bodied tray or syringeable impression material in one-step technique (monophase or double mix) for:

- Impressions for crowns/bridges, inlays/onlays and veneer preparations

- Functional impressions
- Reline impressions

Identium® Light/Identium® Light fast is to be used as syringeable impression material in one-step technique (double mix) for:

- Impressions for crowns/bridges, inlays/onlays and veneer preparations
- Reline impressions
- Impressions for full or partial dentures

Warnings

Do not use Identium® impression materials for two-step putty-wash impression technique or temporary reliner.

Do not use Identium® impression materials with other elastomeric impression materials.

Cautions

Do not use after expiration date (expiration date: see imprint in foil).

Do not leave any residual impression material in the sulcus or oral cavity.

Do not swallow impression material! If swallowed: Consult a medical doctor if problems arise or persist.

Avoid contact with the eyes. If there is accidental contact with the eyes: Rinse immediately and thoroughly with an eye wash or water. Consult a medical doctor if problems arise or persist.

The products do normally not cause allergic reactions; however, for sensitive persons, an allergic reaction to the products cannot be ruled out.

Use of products containing active sulfur, aluminium chloride or nitrogen compounds (retraction cords containing ferric sulfate, polysulfid impression materials, etc.) in conjunction with this product will interfere with the setting reaction of the Vinylsiloxanether® material. Use of these materials requires the area to be rinsed thoroughly to remove all residue. Avoid contact with latex gloves.

Identium® foil bag material: Before fitting the dynamic mixing tip, dispense material until equal amounts of base and catalyst appear; wipe excess. Firmly place dynamic mixing tip and lower locking lever.

When taking impressions of areas with severe undercuts and wide interdental spaces, standard blocking-out techniques must be used.

When using a custom impression tray, ensure that sufficient space remains between the side of the tray and the teeth/jaw.

Do not allow the material to enter the sewer or water system to avoid environmental hazard.

Avoid contact with clothing, as the material cannot be removed by dry cleaning.

Product Overview

Product name	ISO 4823	Consistency (approx.) mm	Mixing ratio and total content volume per unit	Mixing technique	Mixing element
Identium® Heavy	Type 1, Heavy-bodied	32	5:1, 362 ml in foil bags	Plug & Press® Dispenser or other automatic dispensing and mixing system	dynamic mixer
Identium® Heavy fast	Type 1, Heavy-bodied	32	5:1, 362 ml in foil bags	Plug & Press® Dispenser or other automatic dispensing and mixing system	dynamic mixer
Identium® Medium	Type 2, Medium-bodied	36	5:1, 362 ml in foil bags	Plug & Press® Dispenser or other automatic dispensing and mixing system	dynamic mixer
Identium® Medium fast	Type 2, Medium-bodied	36	5:1, 362 ml in foil bags	Plug & Press® Dispenser or other automatic dispensing and mixing system	dynamic mixer
Identium® Medium soft	Type 2, Medium-bodied	36	5:1, 362 ml in foil bags	Plug & Press® Dispenser or other automatic dispensing and mixing system	dynamic mixer
Identium® Medium soft fast	Type 2, Medium-bodied	36	5:1, 362 ml in foil bags	Plug & Press® Dispenser or other automatic dispensing and mixing system	dynamic mixer
Identium® Light	Type 3, Light-bodied	42	1:1, 50 ml cartridge	Applyfix® 4 dispensing gun DS-50 1:1/2:1	static yellow mixing tip MB Ø 4.2 mm
Identium® Light fast	Type 3, Light-bodied	42	1:1, 50 ml cartridge	Applyfix® 4 dispensing gun DS-50 1:1/2:1	static yellow mixing tip MB Ø 4.2 mm

Technical Data

Product name	Mixing ratio	Working time at 23° C/ 74° F ≤	Working time at 35° C/ 95° F ≤ ¹⁾	Intraoral setting time at 35° C/ 95° F ≥	Total setting time* ≥	Hardness (approx.) Shore	Linear dimensional change (approx.) %	Elastic-Recovery test (approx.) %	Strain in compression (approx.) %
Identium® Heavy	5:1	2 minutes	80 seconds	3 minutes 30 seconds	5 minutes 30 seconds	A 60	-0.20	99.0	2.8
Identium® Heavy fast	5:1	1 minute 15 seconds	40 seconds	2 minutes 15 seconds	3 minutes 30 seconds	A 60	-0.20	99.0	2.8
Identium® Medium	5:1	2 minutes	80 seconds	3 minutes 30 seconds	5 minutes 30 seconds	A 60	-0.20	99.0	2.3
Identium® Medium fast	5:1	1 minute 15 seconds	40 seconds	2 minutes 15 seconds	3 minutes 30 seconds	A 60	-0.20	99.0	2.3
Identium® Medium soft	5:1	2 minutes	80 seconds	3 minutes 30 seconds	5 minutes 30 seconds	A 50	-0.20	99.0	3.3
Identium® Medium soft fast	5:1	1 minute 15 seconds	40 seconds	2 minutes 15 seconds	3 minutes 30 seconds	A 50	-0.20	99.0	3.3
Identium® Light	1:1	2 minutes	80 seconds	3 minutes 30 seconds	5 minutes 30 seconds	A 46	-0.20	99.0	3.8
Identium® Light fast	1:1	1 minute 15 seconds	40 seconds	2 minutes 15 seconds	3 minutes 30 seconds	A 46	-0.20	99.0	3.8

*Total setting time (removal time from mouth) from start of mix.

¹⁾With a reduced working time the total setting time of 3 minutes 30 seconds for the fast set version and 5 minutes 30 seconds for the regular set version has always to be heeded.

Note

To ensure optimal impressions the temperature of the material should not deviate from 23° C (74° F) before applying. Otherwise, working and setting times will be affected.

Impression tray: Preparation and adhesives

In principle, all common impression trays can be used if a relevant dynamic pressure is guaranteed. When retention is not sufficient, to ensure a strong bond to the impression material, brush the impression tray with a thin film of Identium® Adhesive prior to loading the tray with impression material. Allow to dry as per instructions for use and cover with a sufficient amount of impression material.

Instructions for use:

Foil bag material for use in most automatic dispensing and mixing systems

Before using the base foil for the first time, remove the white safety pin from the activating head of the large foil bag by turning it in the direction of the arrow and pulling it out (Figure 1).

Place the large foil bag with the integrated activating head in the cartridge body. Ensure that the notches on the activating head and cartridge case are aligned (Figure 2).

Firmly press the activating head into the final position on the cartridge body. The foil bag is automatically pierced by a pin when the activating head is pressed down (Figure 3).

Follow the same procedure as in Figure 2 and 3 for the smaller foil bag. (Note: There is no safety pin on the catalyst foil bag.)

Instructions for use of Kettenbach's automatic dispensing and mixing system (Plug & Press® Dispenser) in combination with the Kettenbach's dynamic mixing tip

It is recommended to use only Kettenbach dynamic mixing tips to ensure optimum performance. If not using a Kettenbach Plug & Press® Dispenser, please follow the manufacturer's instructions for your automatic dispensing and mixing system.

- Move the plungers to the top by turning the control knob (Figure 4).
- Insert the cartridge body into the unit (Figure 5).

- Turn the control knob to move the plungers into the cartridge body and continue turning until the plungers come into contact with the foil bags (Figure 6).
- Once there is contact, press one of the two start keys (at the front on top of the unit) to dispense the material (Figure 7).
- Before fitting the dynamic mixing tip, dispense material until equal amounts of base and catalyst appear (Figure 7), wipe excess. Firmly place dynamic mixing tip (Figure 8) and lower locking lever.

Load the tray with the required amount of material. Hold the impression tray at an angle and press lightly against the tray (Figure 9). Leave the dynamic mixing tip in the material when dispensing. Leave the filled dynamic mixing tip on the cartridge body as a seal.

Before next use, release the locking clip to remove the used dynamic mixing tip and check that the outlets in the activating heads are not blocked. Fit a new dynamic mixing tip, lower locking lever and continue as usual.

When finished simply remove the empty foil bags and activating heads from the cartridge body and discard them. The cartridge body is reusable (Figure 10). The activating heads are disposable.

Instructions for use:

Cartridge material for use in Applyfix® 4 dispensing gun DS-50 1:1/2:1

Insert the cartridge into the Applyfix® 4 dispensing gun DS-50 1:1/2:1. Ensure that the notches on the cartridge base are pointing downwards. The clasp will not close if the cartridge is not properly inserted into the dispensing gun.

Remove the cartridge cap. The cap can be replaced after initial use.

Advance plunger into cartridge and dispense a small amount of impression material until equal amounts are extruded at the same rate.

Install a mixing tip on the cartridge and turn cap 90 degrees in a clockwise direction to lock in place.

Inject the required amount of material, e.g., directly into the tray or the preliminary impression, or into a delivery syringe. To apply the material around the prepared teeth use the delivery syringe or the cartridge with an intraoral tip on the mixer. Check to ensure materials are set prior to mouth removal.

Leave used mixing tip on the cartridge after use or replace the cartridge cap. Before using the cartridge again, remove cartridge cap or remove and discard the used mixing tip.

Check the cartridge openings to ensure no polymerized material is present. Should this be the case, unblock the openings and dispense a small amount of impression material until equal amounts are extruded at the same rate.

Install a new mixing tip and continue as described above.

The Applyfix® 4 dispensing gun DS-50 1:1/2:1 can be sterilized in the autoclave.

Instructions for use:

Applyfix® 5 application syringe

Insert a new syringe tip into tip retainer. Screw tip retainer with the syringe tip onto the syringe cylinder. (Figure II)

If necessary, lubricate the ring washer very slightly with petrolatum for an easy action of the syringe piston.

Prepare the preferred impression material (the Applyfix® 5 application syringe is indicated for light-bodied or medium-bodied consistency impression material only).

Attach the open syringe cylinder of the Applyfix® 5 directly to the mixing tip of the preferred foil bag impression material. (Figure III)

Fill the needed amount of material directly into the syringe cylinder. Insert the syringe piston into the syringe cylinder (Figure IV) and apply the impression material into the patient's mouth as desired.

Cleaning

Clean the Applyfix® 5 application syringe using with the brush (Figure V) after using and removing of the full hardened impression material. The Applyfix® 5 can be reused and autoclaved several times and should be replaced once it shows signs of mechanical damage. Do not reuse the syringe tips. To avoid contamination.

Cautions

Only use one material per Applyfix® 5 application syringe. Contamination of different impression materials renders the impression material unusable.

Note

Keep in the original container.
Selected Kettenbach materials/products are available in certain markets only.

Storage conditions:

Store in a dry place at room temperature; do not expose to sunlight and avoid exposure to extreme temperature fluctuations.

Disinfection:

Impressions can be disinfected using for example, a 2% acidic glutaraldehyde solution. Use disinfection solutions-specific for impression materials (e.g., Silosept®). See manufacturer instructions.

Electroplating:

Impressions may be silver-plated with an alkaline-plating solution only.

Model Pouring:

Impressions with Identium® may be poured immediately after disinfection of the impression up to several weeks with high quality dental stone (e.g., Tewelock®, Twestone®). Impressions can also be poured several times.

Trademarks

Kettenbach, Applyfix® 4, Applyfix® 5, Identium®, Silosept®, Tewelock®, Twestone® and Plug & Press® Dispenser are registered trademarks of Kettenbach GmbH & Co. KG.

Limitation of liability

Except where prohibited by law, Kettenbach GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence, or strict liability.

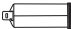
The information provided for Kettenbach products is based on comprehensive research and experience in application technology.

Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development.

However, users must comply with and consider all recommendations and information in connection with any use.

Updated: 15th March 2010

Legend

Conformity with MDD 93/42/EEC		Use by	
Temperature limitation		Keep dry	
Keep away from sunlight		Catalog number	REF
Batch code		Caution, consult accompanying documents	
Dual system (Germany only)		Diameter	
Cartridge		Static mixing tip MB (single use)	
Mixing tip Type B	MB	Intraoral tip (single use)	
Adhesive		Single use	
Cartridge body		Foil bags	
Dynamic mixing tip (single use)		Rx only For professional use only. Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a dentist.	
Applyfix® 5 Syringe tips		Applyfix® 5 application syringe (single use)	
Millimeter	mm	Milliliter	ml
Less than or equal to	≤	Greater than or equal to	≥

- Identium**[®] Heavy
- Identium**[®] Heavy fast
- Identium**[®] Medium
- Identium**[®] Medium fast
- Identium**[®] Medium soft
- Identium**[®] Medium soft fast
- Identium**[®] Light
- Identium**[®] Light fast

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