### **Calasept**<sup>°</sup> Endoline

CALASEPT®Plus Calcium Hydroxide

1,5 ml

Directa's products are developed and evaluated by a selected team of highly-qualified and renowned Swedish and international dentists. It is our goal to make life easier for dental professionals in their daily clinical work.

DIRECTA

### Long-lasting and very effective

The very high concentration of >41% calcium hydroxide, in combination with optimal calcium ion release, creates an efficient and long-lasting antimicrobial effect.

## Direct, quick and easy application

Calasept is a ready-to-use paste available in an airtight syringe, allowing for direct application through the needle. The airtight syringe eliminates the risk of incorporating air into the calcium hydroxide, turning the material into calcium carbonate.

# Optimal calcium release

Made of pure calcium hydroxide. The high concentration of calcium hydroxide creates a long-lasting effect, with the paste releasing calcium ions long after initial placement.

## One product – three indications

Can be used for temporary root fillings, direct pulp capping and indirect pulp capping in deep cavities.





### Calasept® Calcium Hydroxide

Calasept has a very high calcium hydroxide content (>41%), allowing the hydroxyl ions present to generate an elevated pH value of 12.4. This ensures that the paste has a pronounced antimicrobial effect. The high concentration of calcium hydroxide also means that Calasept is long-lasting, releasing calcium ions long after placement. Calasept is a ready-to-use paste in an airtight syringe, applied directly through the needle.

Prod No	
1230200	
1230100	
1231100	

#### Product Description

Calasept 2U, 2 syringes à 1,5 ml Calasept 4U, 4 syringes à 1,5 ml Calasept Paste Needle, 100 pcs





### **Calasept<sup>®</sup> Plus Calcium Hydroxide**

The consistency of Calasept Plus allows calcium hydroxide to be applied using a small and flexible plastic needle, Flexi-Tip. The autoclavable Flexi-Tip in combination with the smooth consistency is time saving and helps for precise and deep application.

Prod No 1240100

**Product Description** Calasept Plus 4U, 4 syringes à 1,5 ml 1241100 Calasept Plus Flexi-Tip, 100 pcs

### Long-lasting and very effective

The very high concentration of >41% calcium hydroxide, in combination with optimal calcium ion release, creates an efficient and long-lasting antimicrobial effect.

### **Smoother** consistency

Calasept Plus smooth consistency allows for direct placement of calcium hydroxide in the root canal through Flexi-Tip, a small and flexible plastic

### **Precise and deep** application with Flexi-Tip

The autoclavable Flexi-Tip, in combination with the smooth consistency, is time-saving, allowing for precise and deep

### **One product – three** indications

Multi-purpose, Calasept Plus can be used for temporary root fillings, direct pulp capping and indirect pulp capping in deep cavities.

## Increased adhesion to filling materials

Removes smear layer. The EDTA forms a water soluble chelate complex that decalcifies substances and dissolves smear layers, thereby increasing adhesion to filling materials.

### Easy filling of syringe

The bottle neck design allows for simplified filling of the syringe.



### Calasept<sup>®</sup> EDTA irrigation agent for root canal walls chelating and smear layer removal

Calasept EDTA has a lubricating capacity and is specially formulated to aid in the negotiation of root canals with endodontic instruments. The EDTA forms a water soluble chelate complex that decalcifies substances and dissolves smear layers, thereby increasing the adhesion of filling materials. During endodontic treatment, EDTA and sodium hypochlorite (NaOCI) are generally used intermittently. The root canal should be thoroughly rinsed with sterile water or saline solution after use of EDTA. Calasept CHX is recommended for final rinsing.

CALASEPT EDT

10

Prod NoProduct Description1260100Calasept EDTA, 100 ml





#### Calasept<sup>®</sup> CHX for irrigation during endodontic treatment

Calasept CHX is a 2% chlorhexidine digluconate with excellent antimicrobial activity. Highly bactericidal, 2% CHX is effective against both Gram-negative and Gram-positive bacteria and is the only irrigating agent that has a high kill rate against E faecalis. Therefore, irrigation with Calasept CHX solution prior to filling a canal with root filling material is highly recommended. CHX is a very reactive compound and will form precipitates with both EDTA and NaOCI. Always rinse the canal with sterile water or saline solution before applying CHX to the canal.

Prod NoProduct Description1270100Calasept CHX, 100 ml

## Inhibitory capacity

Inhibits the growth of bacteria in root canals and dentin tubules.

## Final rinsing

Calasept CHX is recommended to use for final rinsing of the root canal after use of EDTA.

## Easy filling of syringe

The bottle neck design allows for simplified filling of the syringe.





### Calasept<sup>®</sup> Temp self curing paste to be used as a temporary filling.

Calasept Temp is an Eugenol free temporary filling material that is ready to use an requires no mixing. The material is rapidly self curing in the cavity and has a good adhesion to dentin.

- Compositions:
- Zink oxide: 15%
- Glycerine: 20%
- Calcium sulphate hemihydrate: 50%

Prod NoProduct Description1280000Calasept Temp, 2x 30 g



### Calasept<sup>®</sup> Irrigation Needles double-sided vent for efficient irrigation during root canal therapy

This high quality, double-sided vent, luer lock irrigation needle ensures effective and safe irrigation when performing endodontic treatment. The double-sided openings are designed to optimize cleaning of the canals, creating a "swirl-effect". The rounded and closed tip make them safe to use.

## Prod No Product Description 1201600 Calasept Irrigation Needle 27G, Grey, 25 pcs needles 1201610 Calasept Irrigation Needle 31G, Violet, 25 pcs needles

### Calasept® & Calasept® Plus

Calasept Plus and Calasept are composed of pure calcium hydroxides. The very high concentration of >41% calcium hydroxide, in combination with optimal calcium ion release, creates an efficient and longlasting antimicrobial effect. Calasept Plus and Calasept are radio-opaque.

Calasept is a ready-to-use paste available in an airtight syringe, allowing for direct application through the paste needle or Flexi-Tip.

The airtight syringe eliminates the risk of incorporating air into calcium hydroxide, turning the material into calcium carbonate. Calcium carbonate cannot be re-wet into calcium hydroxide.

#### INDICATIONS

Temporary root-filling. Direct pulp capping and indirect pulp capping in deep cavities.

#### **DID YOU KNOW?**

Calasept has a bactericidal effect on carious dentine.

Calasept creates a defensive response, promoting formation of hard tissue barriers.

Calasept stored in the refrigerator will not prolong its shelf-life, it only results in making the consistency thicker.







#### Conclusions

A significant difference in release of calcium ions was observed between setting and pure materials. Calasept showed the highest quantity of calcium release. The values were obtained after 7 days, which is a crucial time when closing the pulp. Values can vary due to draining of molecule content in this study. Dr. Prof. Paz Alejandro

University J F Kennedy Argentina, University la Plata.

Argentina 2007/2008

#### Temporary root-filling



The radiograph of 35 with periapical periodontits.



Two cavities after pulpotomy of two small lesions, showing haemostasis in the mesial, first prepared cavity (arrow) and cessation of bleeding in the distal cavity.



After gross excavation, the deepest layer of carious dentine will be covered with Calasept.



3 months later the periapical radiolucency has been replaced with new bone promoted by the Calasept.



Appearance of hard-tissue barrier 3 months after pulpotomy.



After 6 weeks the cement was removed, Calasept washed away and the excavation of the carious dentin completed without pulp exposure.

Indriect pulp capping

#### Direct pulp capping



### **Calasept**<sup>®</sup>

Calcium Hydroxide

"The original!" **CALASEPT®** 

### Design By Dentists<sup>™</sup>

Directa's products are developed and evaluated by a selected team of highlygualified and renowned Swedish and international dental professionals with the aim of making life easier for dental professionals in their daily clinical work.

# DIRECTA

**DIRECTA AB** Finvidsväg 8, 194 47 Upplands Väsby, SWEDEN +46 8 506 505 75

#### DIRECTA INC

64 Barnabas Road, Unit 3 Newtown, CT 06470, United States 203,491,2273

**DIRECTA GERMANY** Hammergutstr. 11 92245 Kümmersbruck, GERMANY

info@directadental.com www.directadental.com www.directadentalgroup.com www.facebook.com/directadental www.instagram.com/directadental www.youtube.com/user/DirectaAB

DirectaDentalGroup DIRECTA topdental ORSING parkell physics RØNVIG TrollDental Sendoline CONTACEZ

2022-02-24\_REV02

CALASEPT®Plus Calcium Hydroxide 1,5 ml

19004