CHX ENDO (EN)

Description:

CHX Endo 0,2% is a chlorhexidine digluconate solution. It is intended for the irrigation and debridement of root canals during and after instrumentation in endodontic treatment. Remove smear layer with appropriate irrigant such as EDTA solution or Citric Acid 12,5%. For professional use only.

Instructions for use:

1. Use 17% EDTA as a pre-rinse in the canal to remove smear layer.
2. Rinse with Canasol Sodium Hypochlorite (NaOCl).
3. Rinse out the NaOCl with sterile water or anesthetic solution.
4. Fill syringe with CHX Endo 0,2% (10 ml recommended).
5. Continuously irrigate for a minimum of 2 minutes (10-20ml per canal recommended).
6. Remove and dispose of syringe and irrigating tip.

Precautions:

1. Read MSDS for material handling and safety information.
2. Irrigation should not be irritant for the apical area.
3. Not to be used as an oral rinse.

Warnings:

1. Products containing chlorhexidine must not be mixed with products containing sodium hypochlorite to prevent formation of a reddish-brown low solubility precipitate.
2. Keep out of the reach of children.
3. Avoid contact with skin and eyes.

Storage:

Store at room temperature, dry area, away from heat and direct sunlight.

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