
DE	Gebrauchsanweisung.....	5
GB	Instructions for use	9
FR	Mode d'emploi	13
ES	Instrucciones de uso	17
IT	Istruzioni per l'uso	21
PT	Instruções de uso	25
NL	Gebruiksaanwijzing	29
SE	Bruksanvisning	33
DK	Brugervejledning	37
NO	Bruksanvisning	41
FI	Käyttöohjeet	45
GR	Οδηγίες χρήσης	49
CZ	Návod k použití	53
HU	Használati utasítás	57
LV	Lietošanas instrukcija.....	61
LT	Naudojimo instrukcija	65
PL	Instrukcja obsługi	69
RU	Инструкция по применению	73

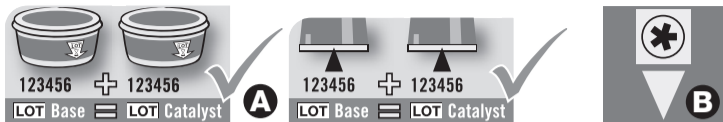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Elastomeric, silicone-based precision impression material, addition-cured

All products in the Provil *novo* assortment have a dosing ratio 1 : 1 (volume or weight).

Practical tips

Provil novo assortment is a variable consistency system that can be combined individually for all impression taking techniques. We do not recommend use in combination with any other silicone impression material. Visually inspect for damage before first use. Damaged products must not be used. Do not swallow and ingest. If health problems arise after swallowing impression material, seek medical attention immediately. Intestinal blockage may arise in rare cases. No allergic reactions have been demonstrated; however the product may still generate an allergic reaction. For use by dental personnel only.

Tubes and Tubs must be carefully sealed after use. Do not inter-change lids, caps or spoons. When using cartridges extrude some material before positioning mixing tip. After use leave used mixing tip in situ as sealer on cartridge. Before next use remove sealed tip. Extrude some material and then fix new mixing tip.

As is common with all addition cured silicone impression materials inhibiting reactions with catalyst paste can occur when in contact with latex gloves, retraction cords or solutions and newly placed composite fillings. Therefore, please ensure that patient's mouth (i.e. preparation and gingival margin) is thoroughly washed and dried before impression taking commences.

	EN ISO 4823	Product colour		Mixing time
		Base	Catalyst	s
Putty <i>fast set</i>	Type 0 putty	green	light grey	30
Putty <i>regular set</i>		green		30
Putty soft <i>fast set</i>		turquoise		30
Putty soft <i>regular set</i>		turquoise		30
Monophase <i>Tube</i>	Type 2 medium bodied	blue	light grey	30
Monophase CD 2 <i>fast set</i>		blue		–
Monophase CD 2 <i>regular set</i>		light blue		–
Medium <i>Tube</i>	Type 2 medium bodied	yellow	light grey	30
Medium CD 2 <i>fast set</i>		yellow		–
Medium CD 2 <i>regular set</i>		light yellow		–
Light <i>Tube</i>	Type 3 light bodied	dark green	light grey	30
Light CD 2 <i>fast set</i>		dark green		–
Light CD 2 <i>regular set</i>		green		–

*at room temperature of 23°C (73°F), 50% rel. humidity.

Higher temperatures shorten, lower temperatures prolong these times.

Extraoral working time* 23°C (73°F) min:s	Intraoral working time 35°C (95°F) min:s	Min. time in mouth min:s	Accessories	
			Mixing Tips	Intraoral Tips
1:30	–	2:30	–	–
2:00	–	3:30	–	–
1:30	–	2:30	–	–
2:00	–	3:30	–	–
2:30	1:00	3:30	–	–
1:30	0:40	2:30	green 66001989	white 65687514
2:00	1:00	3:30	green 66001989	white 65687514
2:30	1:00	3:30	–	–
1:30	0:40	2:30	yellow 66001988	yellow 66000782
2:00	1:00	3:30	yellow 66001988	yellow 66000782
2:30	1:00	3:30	–	–
1:30	0:40	2:30	yellow 66001988	yellow 66000782
2:00	1:00	3:30	yellow 66001988	yellow 66000782

Side effects:

This product or one of its components may in particular cases cause hypersensitive reactions. If suspected, information on the ingredients can be obtained from the manufacturer.

Model fabrication:

The impression can be cast 60 minutes after removal from the mouth – there are no other time constraints. The impression is stable in long-term storage.

Recommended Model material:

All standard model materials can be used.
We recommend:
Class III Gypsum, e. g. MOLDANO®;
in the U.S.: Modern Materials®, Die Keen®;
Class IV Gypsum, e. g. MOLDASTONE®/MOLDASYNT®

Electroplating:

Impressions can be electroplated with copper or silver in conventional baths.

Disinfection:

Impressions can be treated with an aqueous sodium hypochlorite solution (5.25%, for up to 10 minutes*) or with other disinfectant solutions suitable for silicones. When doing so, please observe the disinfection/processing instructions issued by the relevant manufacturers.
* (DGZMK 3/93 V1.0)

Adhesion:

We recommend using our Universal Adhesive.

Storage conditions:

Do not store above 25°C (77°F).

Dated: 2014-03

Description of applicational/ packaging icons

A = Only use pastes with same LOT No
B = Icon "Use of the special program at the Dynamix® speed Mixing Unit possible"
C = Mixing ratio
D = Mixing time

E = Extraoral working time
F = Intraoral working time
G = Time in mouth
H = Static mixing tip
K = Intraoral tip

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Kulzer GmbH
Kulzer, LLC