Honigum Pro

Mono

Inhalt | Contents

Gebrauchsinformation – DEUTSCH	4
Instructions for use – ENGLISH	14
Mode d'emploi – FRANÇAIS	24
Istruzioni per l'uso – ITALIANO	35
Instrucciones de uso – ESPAÑOL	46
Instruções de utilização – PORTUGUÊS	57
Gebruiksaanwijzing – NEDERLANDS	68
Brugsanvisning – DANSK	78
Bruksanvisning – SVENSKA	88
Instrukcja użycia – POLSKI	98
Инструкция по применению – РУССКИЙ	109

Produktbeschreibung

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Vinylpolysiloxane, Siliciumdioxid, Pigmente, Additive, Platinkatalysator

Klassifikation

ISO 4823:2000, Type 2: Medium-bodied consistency

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Product description

Honigum Pro/Mono is a monophasic impression material based on addition curing silicones.

The material is available in the Automix safety cartridge and in the ready-to-use pre-filled MixStar cartridge, which is suitable for use in the DMG MixStar-eMotion and other commercially available mixing machines.

Indications

- Crown and bridge impressions
- Inlay and onlay impressions
- Functional impressions
- Pressure point identification
- All types of pickup impressions, e.g. for implants and telescopes

Basic safety instructions

- For dental use only!
- Keep away from children!


Advice on application

- The working times (refer to “Technical data” section on page 20) apply at a room temperature of 23 °C and a normal relative air humidity of 50%. Higher temperatures shorten, lower temperatures increase these times.
- Rinse off residues of retraction solutions thoroughly with water because they can increase the setting time in some instances.
- Never allow the impression to come into contact with organic solvents or liquids containing solvents as this may result in a swelling of the material and may therefore lead to inaccuracies in the cast.
- Never allow the impression material to come into contact with latex gloves. DMG recommends the use of PVC or PE based gloves.


- Do not mix Honigum Pro/Mono with condensation curing silicone or polyether impression materials.

Using the MixStar cartridge in the DMG MixStar-eMotion


- ▶ Follow the instructions for use for the “MixStar-eMotion”!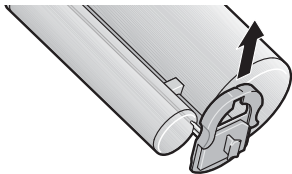


1. Slide the locking mechanism upwards in order to open the cartridge. Do not remove the locking mechanism from the cartridge.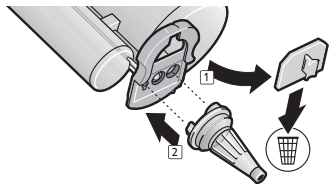

2. Remove the cap from the cartridge and discard. Attach a new mixing tip.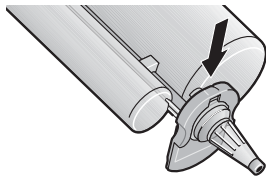

3. Secure the mixing tip by sliding the locking mechanism into the locked position.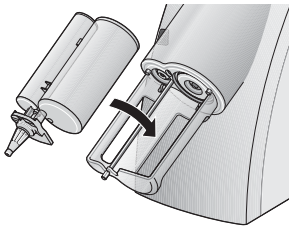

4. Insert the cartridge into the MixStar-eMotion. A brief acoustic signal confirms the correct insertion of the cartridge.

Note: Prepare fresh cartridges for use by extruding and discarding the first 3 cm of impression material.

Leave the used mixing tip on the cartridge after use. It serves as a cap.

Using the MixStar cartridge in mixing units from other manufacturers

- ▶ Follow the instructions for use for the chosen mixing unit!
- ▶ First insert the cartridge into the mixing unit without the mixing tip. Otherwise appropriate mixing of the material cannot be guaranteed!
- ▶ In this case proceed as described in the section “Changing the mixing tip” section on page 16.

Changing the mixing tip


- ▶ Follow the instructions of the mixing unit you are using!

Note: Attach a new mixing tip before each mixing process.

1. Slide the locking mechanism upwards.
2. Remove the old mixing tip and any hardened material blocking the cartridge openings.

3. Place the mixing tip securely on the respective cartridge opening.
4. Lock the mixing tip in place with the locking mechanism.

Using the Automix cartridge

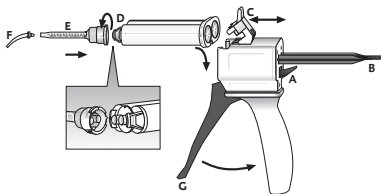Insert cartridge

1. Push up the lever [A] at the back of the dispensing gun and pull back the slide [B] completely.

2. Lift up the plastic lever [C], insert the cartridge and lock it by pushing down the plastic lever [C].

Note: Make sure that the notches on the cartridge line up with those on the dispensing gun.

Attach mixing tip

1. Remove the cartridge cap [D] or used mixing tip by turning it 90° counterclockwise, and discard it.
2. Attach a new mixing tip [E].

Note: Make sure that the notches on the mixing tip line up with those on the cartridge.

3. Turn the mixing tip clockwise 90° to lock it.

Apply material

- ▶ When using a cartridge for the first time, express a pea-sized amount of material and discard it.
- ▶ The material is mixed in the mixing tip by pulling the trigger [G] of the dispenser and it is then applied directly.

Note: Leave the used mixing tip on the cartridge after use. It serves as a cap.

Remove cartridge

1. Push up the lever [A] at the back of the dispensing gun and pull back the slide [B] completely.
2. Lift up the plastic lever [C] and remove the cartridge.

Recommended use

- Honigum Pro/Mono can be used in all common nonperforated impression trays.
- Best results are obtained with individual impression trays.
- In order to improve the adhesion of the material to metal or plastic trays, a tray adhesive for A-silicon (e.g. Tray Adhesive, DMG) should be applied.

Monophase impression taking

- ▶ For the individual times refer to the “Technical data” section on page 20.

- ▶ Fill the prepared tray with Honigum Pro/Mono. When doing so, hold the impression tray at a slight angle beneath the mixing tip.

Note: To avoid air bubbles, the end of the mixing tip should stay immersed in the impression material throughout the process.

- ▶ Fill a small amount of material into a commercially available elastomer syringe or use the 50 mL cartridge directly and syringe the preparation.

Note: The filling of the tray and syringing should be completed at the same time as the higher temperature in the mouth leads to a quicker setting of the syringed material compared to the tray material.

- ▶ Position the filled tray in the mouth within the working time, slowly and using very little force, and allow the impression to form.
- ▶ Remove the impression from the mouth once the setting time has ended, refer to “Removal” section on page 19.

Double-mix impressions

- ▶ For the individual times applicable to the materials used refer to the “Technical data”. Follow the associated instructions for use!
- ▶ Fill the tray with Honigum Pro/Mono. When doing so, hold the impression tray at a slight angle beneath the mixing tip.

Note: To avoid air bubbles, the end of the mixing tip should stay immersed in the impression material throughout the process.

- ▶ Syringe the preparation with Honigum Pro/Light.

Note: The filling of the tray and syringing should be completed at the same time as the higher temperature in the mouth leads to a quicker setting of the syringed material compared to the tray material.

- ▶ Position the filled tray in the mouth within the working time, slowly and using very little force, and allow the impression to form.

- ▶ Remove the impression from the mouth once the setting time has ended, refer to “Removal” section on page 19.

Removal

- ▶ Remove the impression parallel to the longitudinal axis of the prepared teeth.
- ▶ After removal, proceed as described in the “Cleaning and disinfecting the final impression” section on page 19.

Cleaning and disinfecting the final impression

- ▶ Only disinfect the final impression with dipping solutions. DMG recommends the use of a 2% glutaraldehyde solution.
- ▶ Please ensure that the disinfectant used is indicated for use with addition curing silicone impression material.

- ▶ Follow the instructions of the disinfectant solution's manufacturer precisely!
- ▶ Leave the final impression to dry and store until casting, refer to "Storage of the impression" section on page 20.
- ▶ For more information, refer to the comments of the DGZMK. Comments of the DGZMK 3/93 V 1.0; www.dgzmk.de
- ▶ ADA recommends following Council on Dental Materials, Instruments and Equipment. Disinfection of impressions. JADA 1991;122(3):110; www.ada.org

Casting and galvanization

- ▶ The impression should be cast 30 minutes at the earliest after removal from the mouth.

DMG recommends using special purpose hard plaster (ISO 6873, Type 3; e.g. Moldano, Bayer) for the cast and an ultra hard dental plaster for the forms (ISO 6873, Type 4; e.g. Fujirock II, GC). The impression can be copper or silver-plated.

Storage of the impression

- ▶ Store the impression in a dry place at room temperature (15–25 °C/59–77 °F).
- ▶ Protect against exposure to direct sunlight!

The impression is stable and can be cast even after months without loss of accuracy.

Technical data

Honigum Pro/Mono		
Working time ¹ [min]	≤ 2:45	≤ 2:45
Intraoral working time ² [min]	≤ 1:00	≤ 1:00
Time in the mouth [min]	≥ 3:15	≥ 3:15
Dimensional change after 24h	≤ 0.40	≤ 0.40
Compression set [%]	≤ 0.35	≤ 0.35
Strain in compression [%]	≈ 2.8	≈ 2.8

Honigum Pro/Mono	 Fast	 Fast
Working time ¹ [min]	≤ 1:30	≤ 1:30
Intraoral working time ² [min]	≤ 0:30	≤ 0:30
Time in the mouth [min]	≥ 2:00	≥ 2:00
Dimensional change after 24h	≤ 0.40	≤ 0.40
Compression set [%]	≤ 0.35	≤ 0.35
Strain in compression [%]	≈ 2.8	≈ 2.8

¹The times apply at a room temperature of 23 °C and a normal relative air humidity of 50%. Higher temperatures shorten, lower temperatures increase these times.

The working time describes the period from the start of mixing until the time by which the tray must be positioned in the patient's mouth.

²The times apply from start of mixing time for a temperature of 35 °C.

Disinfection of the Automix-Dispenser

The Automix-Dispenser is reusable after disinfection, although it should be replaced if visible signs of wear appear.

Disinfection can be carried out using a commercially available dipping disinfectant. DMG recommends using only disinfecting solutions listed by the RKI (Robert Koch Institute). The Automix-Dispenser can also be autoclaved.

Composition

Vinyl polysiloxanes, silicon dioxide, pigments, additives, platinum catalyst

Classification

ISO 4823:2000, type 2: Medium-bodied consistency

Storage and shelf-life

- ▶ Store in a dry place at room temperature (15–25 °C/59–77 °F).

- ▶ Do not store in the refrigerator or freezer!
- ▶ Protect against exposure to direct sunlight!
- ▶ Leave the used mixing tip on the cartridge after use. It serves as a cap
- ▶ Do not use after the expiration date!

Available in the following forms

Not available in the USA

1 cartridge with 380 ml paste, 10 MixStar-Tips	
Honigum Pro/Mono	
	REF 929568
 Fast	REF 929978

5 cartridges with 380 ml paste, 50 MixStar -Tips	
Honigum Pro/Mono	
	REF 929569
 Fast	REF 929979

2 cartridges with 50 ml paste, 4 Automix-Tips, 4 Intraoral-Tips	
Honigum Pro/Mono	
	REF 929830
 Fast	REF 929976

8 cartridges with 50 ml paste, 16 Automix-Tips, 16 Intraoral-Tips	
Honigum Pro/Mono	
	REF 929834
 Fast	REF 929977

8 cartridges with 50 ml paste

Honigum Pro/Mono

REF 929911

Only available in the USA

1 cartridge with 380 ml paste, 10 MixStar-Tips

Honigum Pro/Mono

REF 989568

4 cartridges with 50 ml paste,
8 Automix-Tips, 8 Intraoral-Tips

Honigum Pro/Mono

REF 989830

Caution: US Federal Law restricts this device to sale by, or on the order of a dentist, or other practitioner licensed by law of the state in which he or she practices to use or order the use of this device.

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DMG Chemisch-Pharmazeutische Fabrik GmbH